

Zynex Pro Hybrid LSO 637

The Zynex Pro Hybrid LSO637 TM Brace provides support for both spinal surgeries and conservative treatment of spinal injuries. The brace features universal sizing, an ice pack for optimal patient experience, and multiple L code configurations. The Zynex Pro line features our signature M shaped handle.



Sizing

Zynex Pro Hybrid LSO 637 can be adjusted from 25" to 68". Zynex Pro Hybrid LSO 637 comes with a main panel and two adjustable belts with comfort pull rings. Simply place the brace on a table with the inside facing up. When using each matching number, the waist is measured as the following:

S & S = 25" - 32" waist	2XL & 2XL = 50" - 56" waist
M & M = 32" - 38" waist	3XL & 3XL = 56" - 62" waist
L & L = 38" - 44" waist	4XL & 4XL = 62" - 68" waist
XL & XL = 44" - 50" waist	



Step 1 Match the size using the chart to fit your waist match your belts size. Remove Hook & Loop from belts and pull the tabs through to the sizing creases, securing the sizing flap with Hook & Loop to your appropriate crease 1,2,3,4,5,6,7. For your appropriate size, see chart above and reference **Sizing** box to the left (in blue).

Features and Benefits


- Universal size with Low-profile design
- Simple One Pull Compression System
- Ice Pack
- Three Location Gear System
- Multiple L Code Configurations

Indications for Use


- | | |
|-----------------------|-------------------|
| Low Back Pain | Laminectomy |
| Osteoporosis | Discectomy |
| Disc Degeneration | Spinal Stenosis |
| Disc Herniation | Spondylolisthesis |
| Post-Operative Fusion | |

Ice Pack


Fits easily in the brace, attaches with Hook & Loop.

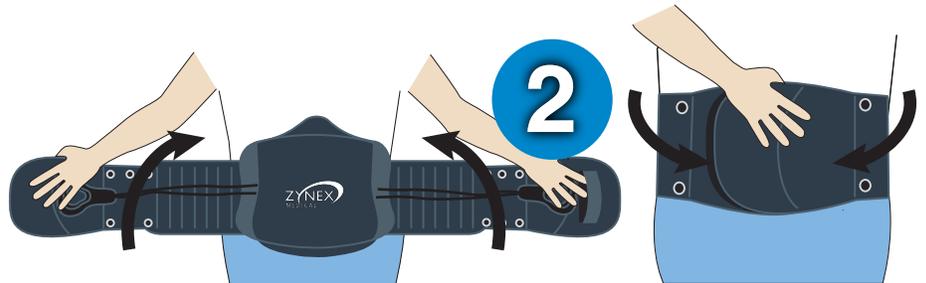

Step 2 Put the brace around your waist and secure the front panel onto the brace so the Hook & Loop secures the fit.

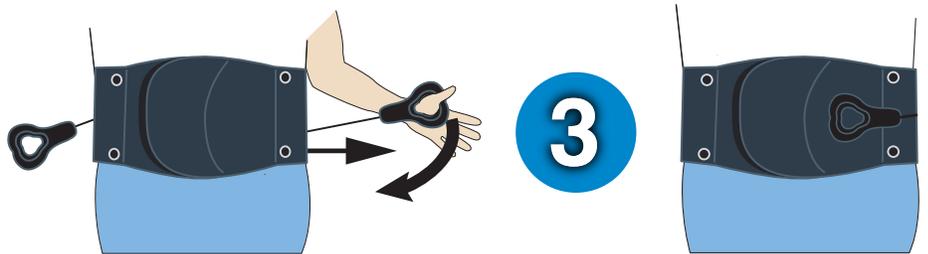

Step 3 Pull both comfort rings forward until you feel comfortable and then secure the rings to your brace by placing them on to the sides. The comfort rings control the compression of the brace and can be used to adjust the fit.

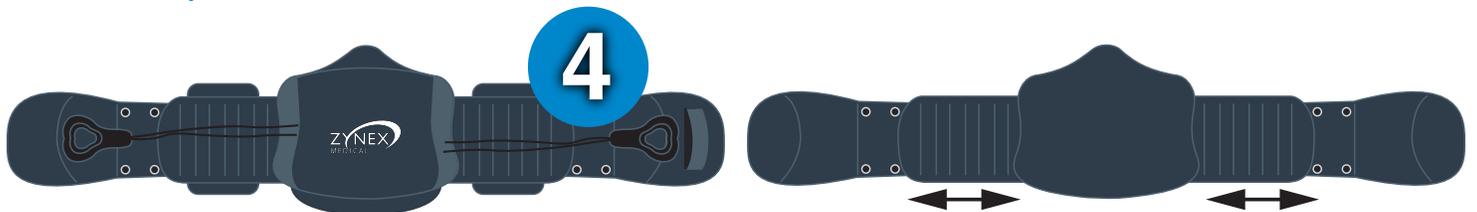

Step 4 (optional) Optional Lateral panels. If you are instructed to use the Lateral panels, secure them to inside of the brace by placing them next to the main panel, with the large curve resting next to the main panel on both sides.

Removing the brace Lift both comfort rings off the brace placing them to the sides and securing them so there is no tension on the brace. Then remove the Hook & Loop connection on the front panel and safely remove the brace.

Order Information

Part Number: 123237

How to order/ technical Support

Size Universal

800 495-6670

Zynex Medical
9655 Maroon Circle
Englewood, CO, 80112
www.Zynex.com

